

GETINGE
GETINGE GROUP

**GETINGE WD15 CLARO
THE CLEAR CHOICE IN
WASHER-DISINFECTORS**



Always with you



INSIGHT IS WHAT MAKES THE DIFFERENCE

Every time we develop a new product, we base our design work on our deep insight into our customers' everyday reality. It's a successful approach, and it has made us the world leader in sterilization and disinfection.

When we designed the new Getinge WD15 Claro our focus was clear: how can we create the optimal under-counter washer-disinfector for wards, clinics and dental practices? How can we ensure the best possible efficiency, ergonomics and ease-of-use?

The result is a high-capacity, fast-working and – if we may say so ourselves – a rather nice-looking little machine called Claro.

The nickname comes from maybe its most distinguishing feature, which actually represents another kind of “insight” – that you have a clear view of what happens throughout the decontamination process, thanks to the glass window and the chamber light.

ALL THE CAPACITY YOU NEED IN 600 MM

Claro is perfectly suited for dental practices where a compact under-counter machine is often the ideal choice to save countertop space. Of course, it combines very well with Getinge tabletop sterilizers and sealers and there is also a wide range of practical accessories.



Unique chamber design

Thanks to smart positioning of water/air connections, the chamber has a large capacity. The carts offer an almost 100% usable surface.

Clever self-closing system

Thanks to the clever self-closing mechanism for the spray arm docking system, the upper cart, along with its spray arm, can be removed to make space for larger goods, whilst still maintaining the same level of cleaning efficiency.

By a simple click the spray arm is completely opened at both ends simplifying the maintenance of the Claro.





VIP – View-In-Process glass window

For extra safety you have a constant clear view of the process thanks to the glass window and the lighting. It's easy to see by just a glance that the spray wings rotate as they should and that the goods are loaded correctly.

Integrated hot air drying

After rinsing, washing and disinfection phases, there is an extra built-in drying phase with HEPA filter 13 that ensures that the goods are quickly and efficiently dried.

Pump pressure monitoring

For extra safety, the circulation pump has a pressure sensor that stops the process and generates an error code if the water pressure is not correct.

Easy to operate

The PACS 350 control system is very user-friendly and program settings are easily changed on the operator panel.

Documentation via printer or Netcom

Printer or Netcom with USB connection for detailed process reports is available as an option. Claro can also be connected to T-DOC, Getinge's comprehensive instrument management system, via Netcom.

Clear display

The machine is self-diagnostic. Program, phase, chamber temperature, error codes and remaining cycle time are clearly shown in the display. It's easy to follow the process and it simplifies servicing.

Condenser cooling

Thanks to a smart solution with condenser cooling there is no need to connect to a ventilation system, which makes it less humid in the room and also simplifies installation.

ALL THE ACCESSORIES YOU MAY NEED

Getinge offers a well-adapted range of accessories and there's a solution whatever kind of goods you need to reprocess.

LOWER BASIC CART

With support for larger loads.
Capacity: 2 SPRI II (340x250x70 mm)
or 2 DIN 1/1 (480x250x50).
Free loading height: with upper wash cart 320 mm.
Without upper wash cart 490mm.
Loading area 520x496mm.
Art.no. 503014600



UPPER BASIC CART

Capacity: 2 SPRI II (340x250x70 mm)
or 2 DIN 1/1 (480x250x50).
Free loading height: 110 mm.
Art.no. 503013400



DENTAL INJECTION BAR

With 10 injectors for hand pieces. Five injectors are for hand pieces with an approximate outer diameter of 16 mm, five for 20 mm. A fine mesh filter is included. (Hand pieces shown in pictures are not included)
Art.no. 503043200



DENTAL KIT

With 1 tray rack for 9 dental trays,
1 SPRI III instrument tray and 1 fine
mesh basket with lid for small parts.
(Trays on pictures not included)
Art. no: 502795400



AVAILABLE IN TWO COLORS

The Getinge WD15 Claro is available in blue/stainless steel or in white powder coated. Both variants can be delivered with a 55 cm cabinet for a more ergonomic work height.



TECHNICAL FEATURES

Design	Blue or White
External Dimensions	600 x 620 x 850 mm (H x W x D)
Internal Dimensions	536 x 536 x 520 mm (H x W x D)
Weight	92 kg
Volume	150 L
Chamber/Panels Material	AISI 316L/AISI 304L/galvanized
Communication Ports	RS232 and RS485
Electrical	Single- or 3-phase 50/60 Hz 200V, 208V, 220-230V, 240V or 400V
Heating Element	Water 6.3 kW dryer 2.8 kW
Water Supply	Cold and Hot
Dosage Pumps	2
Detergent Dosage Control	Yes (standard)
Circulation Pump	380L/min
Noise	<60 dB (A)
Standards	EN ISO 15883, MDD, ELT/SEMKO, EMC
Optional	Cabinet 2 extra dosing pumps with flow control Water softener External printer Netcom bluebox T-DOC (instrument management system)



COMPLETE SOLUTIONS FOR INFECTION CONTROL

Getinge is the world's leading provider of solutions for effective cleaning, disinfection and sterilization in the healthcare and life science sectors. We are dedicated to helping our customers provide better care at a lower cost. We do this by offering well thought through and customized solutions. This means that we are with our customers all the way from architectural planning and education to traceability and support – with complete solutions, long-term commitment and global presence. Getinge – Always with you.

Getinge Infection Control AB

P O Box 69
SE-305 05 Getinge
SWEDEN
Phone: +46 10 335 00 00
info@getinge.com

GETINGE
GETINGE GROUP

www.getinge.com

GETINGE GROUP is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of ArjoHuntleigh, GETINGE and MAQUET. **ArjoHuntleigh** focuses on patient mobility and wound management solutions. **GETINGE** provides solutions for infection control within healthcare and contamination prevention within life sciences. **MAQUET** specializes in solutions, therapies and products for surgical interventions and intensive care.