

**GETINGE**  
GETINGE GROUP

**GETINGE WD15 CLARO  
THE CLEAR CHOICE IN  
WASHER-DISINFECTORS**



Always with you



GETINGE

## INSIGHT IS WHAT MAKES THE DIFFERENCE

Every time we develop a new product, we base our design work on our deep insight into our customers' everyday reality. It's a successful approach, and it has made us the world leader in sterilization and disinfection.

When we designed the new Getinge WD15 Claro our focus was clear: how can we create the optimal under-counter washer-disinfector for wards, clinics and dental practices? How can we ensure the best possible efficiency, ergonomics and ease-of-use?

The result is a high-capacity, fast-working and – if we may say so ourselves – a rather nice-looking powerful machine called Claro.

The nickname comes from its maybe most distinguishing feature, which actually represents another kind of “insight” – that you have a clear view of what happens inside of it all through the process, thanks to the glass window and the chamber light.

## JUST THE CAPACITY YOU NEED

Claro is perfectly suited for healthcare units where a compact under-counter machine is often the ideal choice to save countertop space. Of course, it combines very well with Getinge tabletop sterilizers and sealers and there is also a wide range of practical accessories.



### **Takes 6 DIN trays**

Thanks to smart positioning of water/air connections, the chamber is designed to accommodate two DIN trays on each level.



### **Easy to make room for larger goods**

Just remove the upper cart. There are self-closing dockings on the upper spray arm so you don't have to mount any extra cover plates. The spray arms are opened by a simple click on both ends and simplifies the maintenance of the Claro.





Claro is also available with a stainless steel cabinet for a more ergonomic work height. The cabinet is also a perfect place to store detergents and other consumables.

#### **VIP – View-In-Process glass window**

For extra safety you have a constant clear view of the process thanks to the glass window and the lighting. It's easy to see by just a glance that the spray wings rotate as they should and that the goods are loaded correctly.

#### **Integrated hot air drying**

After rinsing, washing and disinfection phases, there is an extra built-in drying phase with HEPA filter 13 that ensures that the goods are quickly and efficiently dried.

#### **Pump pressure monitoring**

As an extra safety, the circulation pump has a pressure sensor that stops the process and generates an error code if the water pressure is not correct.

#### **Easy to operate**

The PACS 350 control system is very user-friendly and program settings are easily changed on the operator panel.

#### **Documentation via printer or Netcom**

Printer or Netcom with USB connection for detailed process reports is available as an option. Claro can also be connected to T-DOC, Getinge's comprehensive instrument management system, via Netcom.

#### **Clear display**

The machine is self-diagnostic. Program, phase, chamber temperature, error codes and time left are clearly shown in the display. It's easy to follow the process and it simplifies servicing.

#### **Condenser cooling**

Thanks to a smart solution with condenser cooling there is no need to connect to a ventilation system, which makes it less humid in the room and also simplifies installation.

## ALL THE ACCESSORIES YOU MAY NEED

Getinge offers a well-adapted range of accessories and there's a solution whatever kind of goods you need to reprocess.



### LOWER BASIC CART

With support for larger loads.  
Capacity:  
2 SPRI II (340x250x70 mm) or  
2 DIN 1/1 (480x250x50).  
Free loading height:  
With upper wash cart 320 mm.  
Without upper wash cart 490 mm.  
Loading area 520x496 mm.  
Art.no. 503014600



### UPPER BASIC CART

Capacity:  
2 SPRI II (340x250x70 mm) or  
2 DIN 1/1 (480x250x50).  
Free loading height: 100 mm.  
Art.no. 503013400



### MIS INJECTION BAR

To be placed in lower basic cart.  
Equipped with a stand for 4 canals,  
3 injector pipes (Ø 4 mm), 2 silicon  
houses Ø 8 mm with 2 male luer  
lock, 2 injector bodies Ø 15 mm  
and 2 injector Ø 20 mm  
Art.no. 503155500



### AN INJECTION BAR

To be placed in lower basic cart.  
Equipped with  
1 injector pipe for breathing bag,  
1 injector pipe for bellow,  
1 injector pipe for hose connections and  
2 injector pipes for 1 double connect  
larynx mask and  
3 connections for hoses with an inner  
diameter of 8–30 mm and maximal  
length of 1 500 mm.  
A support for hoses is also included.  
Art.no. 503301900



### STAND FOR BOWLS 151 MM

Capacity: 12 B8 (Ø 160 x 80 mm)  
Dimension: 494x317x40 mm  
Art.no. 498859700



### STAND FOR BOWLS 111 MM

Capacity: 12 B6 (Ø 125 x 55 mm)  
Dimension: 494x317x40 mm  
Art.no. 498859600



### STAND FOR BABY BOTTLES

Capacity: 36 bottles  
For baby bottles with an opening  
larger than Ø20 mm.  
Art.no. 477071900



### STAND FOR BABY BOTTLES

Capacity: 24 bottles  
For baby bottles with an opening  
larger than Ø20 mm.  
Art.no. 476942300

## AVAILABLE IN TWO COLORS

Claro is available in blue/stainless steel or in white coated stainless steel. Both variants can be delivered with a 55 cm cabinet for a more ergonomic work height.



### TECHNICAL FEATURES

Design	Blue or White
External Dimensions	596 x 850 x 625 mm (W x H x D)
Internal Dimensions	538 x 520 x 534 mm (W x H x D)
Weight	92 kg
Volume	150 L
Chamber/Panels Material	AISI 316L/AISI 304L
Communication Ports	RS232 and RS485
Electrical	Single- or 3-phase 50/60 Hz 200V, 208V, 220-230V, 240V or 400V
Heating Element	Water 6.3 kW dryer 2.8 kW
Water Supply	Cold and Hot
Dosage Pumps	2
Detergent Dosage Control	Yes (standard)
Circulation Pump	380L/min
Noise	<60 dB (A)
Standards	EN ISO 15883, MDD, EN IEC 61010-1, EN 61000-3-2, DVGW
Optional	Cabinet 2 extra dosing pumps with flow control Water softener External printer Netcom bluebox T-DOC (instrument management system)



#### COMPLETE SOLUTIONS FOR INFECTION CONTROL

Getinge is the world's leading provider of solutions for effective cleaning, disinfection and sterilization in the healthcare and life science sectors. We are dedicated to helping our customers provide better care at a lower cost. We do this by offering well-thought-through and customized solutions. This means that we are with our customers all the way from architectural planning and education to traceability and support – with complete solutions, long-term commitment and global presence. Getinge – Always with you.

#### Getinge Infection Control AB

P O Box 69  
SE-305 05 Getinge  
SWEDEN  
Phone: +46 10 335 00 00  
info@getinge.com

# GETINGE

GETINGE GROUP

[www.getinge.com](http://www.getinge.com)

**GETINGE GROUP** is a leading global provider of products and systems that contribute to quality enhancement and cost efficiency within healthcare and life sciences. We operate under the three brands of ArjoHuntleigh, GETINGE and MAQUET. **ArjoHuntleigh** focuses on patient mobility and wound management solutions. **GETINGE** provides solutions for infection control within healthcare and contamination prevention within life sciences. **MAQUET** specializes in solutions, therapies and products for surgical interventions, interventional cardiology and intensive care.